

Identifying Your Router and WPS Button

The 3B Luna devices are WiFi capable. For WiFi to function, your Luna device must be connected to your home's WiFi router. This is a one-time pairing of your CPAP device to your router.

After the initial connection, your Luna CPAP or Auto-CPAP will connect automatically.



There are two ways of establishing the initial connection.

The first is by manual entry of your router's PIN or password. The second, and easiest, manner is to use the router's WPS "Quick Connect" button. Wi-Fi Protected Setup (WPS) is a network security standard that allow users to easily secure a wireless home network, and connect wireless client devices (iPad's, smart phones, printers, etc.) to it, without accessing the router's configuration screens, and without even knowing the wireless network's security password using a simple push button on the router.

Most routers have a WPS button.

Examples of the most popular routers, and the location of their WPS button are diagrammed on the right.

COMMON LOCATION OF WPS BUTTONS




LINKSYS



CISCO

NETGEAR

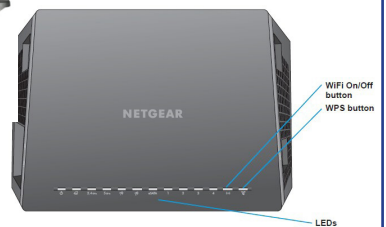

Router WPS buttons often are marked with the following common symbol:

