



3B™ Medical, Inc.

STEPS FOR EASY WI-FI SET-UP

1



Remove the old Module from the back of your machine. Replace with the Wi-Fi module and plug the USB cable into the port on the back of your Luna CPAP.

2



Plug the Luna machine in and allow it to boot up to the home screen.

3



Hold the RAMP button down for several seconds, until the screen changes to the "PATIENT MENU".

4




Turn the dial until the "GEAR" is selected on the side of the menu, push the dial down once.

5



Continue to twist the dial clockwise to highlight "Wi-Fi", select Wi-Fi by pushing the knob once.

6

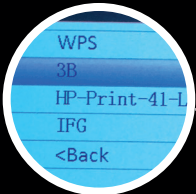

All available networks will show on the screen. Select your network by twisting the dial until it is highlighted, then push the dial once.

WiFi Set Up Continued

If you do not have a password...
CONNECT BY WPS BUTTON

1




Press the WPS button on your router.

2



Press the button on the Wi-Fi symbol at the top of your screen.

3

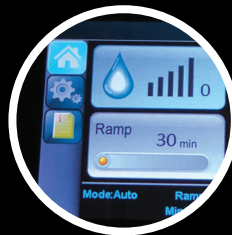

Turn the dial until you have "HOME" highlighted, click once.

CONNECT BY PASSWORD

1



A screen allowing you to scroll through letters, numbers and characters will appear, turn the dial and press once to select each character.

2



When complete, select the arrow at the end of the menu, the Luna will display CONNECTING. You should now see a Wi-Fi signal indicator at the top of your screen.

3



Turn dial until "back" is highlighted, click once. This brings you back to main menu. Turn dial until "home" is highlighted, click once.