



**3B™ Medical, Inc.**

## STEPS FOR EASY WI-FI SET-UP

1



Remove the old Module from the back of your machine. Replace with the Wi-Fi module and plug the USB cable into the port on the back of your Luna CPAP.

2



Plug the Luna machine in and allow it to boot up to the home screen.

3



Hold the RAMP button down for several seconds, until the screen changes to the "PATIENT MENU".

4




Turn the dial until the "GEAR" is selected on the side of the menu, push the dial down once.

5



Continue to twist the dial clockwise to highlight "Wi-Fi", select Wi-Fi by pushing the knob once.

6

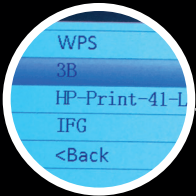

All available networks will show on the screen. Select your network by twisting the dial until it is highlighted, then push the dial once.

# WiFi Set Up Continued

If you do not have a password...  
**CONNECT BY WPS BUTTON**

1




Press the WPS button on your router.

2



Press the button on the Wi-Fi symbol at the top of your screen.

3

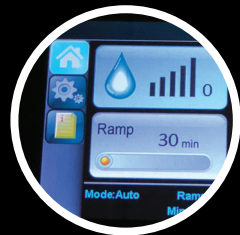

Turn the dial until you have "HOME" highlighted, click once.

## CONNECT BY PASSWORD

1



A screen allowing you to scroll through letters, numbers and characters will appear, turn the dial and press once to select each character.

2



When complete, select the arrow at the end of the menu, the Luna will display CONNECTING. You should now see a Wi-Fi signal indicator at the top of your screen.

3



Turn dial until "back" is highlighted, click once. This brings you back to main menu. Turn dial until "home" is highlighted, click once.