

# Patient's Guide for Luna G3 Devices

**Your New Luna G3 Device** is an important part of your journey in treating your Sleep Disordered Breathing. This guide will assist you in learning more about your Luna G3 from getting started to downloading and interpreting your compliance reports. Please note that this is informational only. Only your Physician and care team can guide you in your care.

You may have a black Luna Device or an ivory Luna Device as shown below. The information provided is applicable for all Luna G3 Models.



Luna G3 APAP (LG3600)



Luna G3 Bilevel 25 A  
(Lg3700), Bilevel ST 30VT  
(LG3800)

## Using Your New Luna G3 Device

### Unboxing

It is normal to feel overwhelmed when starting your therapy. We have created a short video to assist you with unboxing your Luna G3 Device and getting it set up in your home. You can access these videos by clicking the link for your device below:

- » [Luna G3 APAP Device \(LG3600\)](#)
- » [Luna G3 Bilevel 25A Device \(LG3700\)](#)
- » [Luna G3 Bilevel ST 30Vt Device \(LG3800\)](#)

### Getting Started

- » [Getting Started with Your Luna G3 PAP Device](#)
- » [Primeros Pasos Con Su Dispositivo Luna G3: ESPAÑOL](#)

### User Manuals and Quick Start Guides

You should have received a User Manual and Quick Start Guide for your Luna G3. If you did not receive these, or you have misplaced them, you can find electronic versions of them below.

### Patient User Manuals

- » [Luna G3 APAP User Manual](#)
- » [Luna G3 APAP Manual De Usuario: ESPAÑOL](#)
- » [Luna G3 Bilevel 25A User Manual](#)
- » [Luna G3 Bilevel 25A Manual De Usuario: ESPAÑOL](#)
- » [Luna G3 Bilevel ST 30Vt User Manual](#)
- » [Luna G3 Bilevel ST 30Vt Manual De Usuario: ESPAÑOL](#)

### Quick Start Guides

- » [Luna G3 APAP Quick Start Guide](#)
- » [Luna G3 APAP Guía De Inicio Rápido - ESPAÑOL](#)
- » [Luna G3 Bilevel 25A Quick Start Guide](#)
- » [Luna G3 Bilevel ST 30Vt Quick Start Guide](#)

*(Continued on next page)*

(Continued from previous page)

## Compliance and your Luna G3 Device

Compliance to PAP therapy is important to assist you in treating your Sleep Disordered Breathing. Taking an active role in your therapy may include downloading and viewing how compliant you are to your therapy. The Luna G3 PAP devices have several ways to access your compliance data.

## The 3B Luna QR Code Patient App

Your Luna G3 device has an app that can be used to obtain your data. You can find information on how to download and utilize your app below:

- » [Patient Instructional Video for downloading the 3B Luna QR Code App](#)

## Patient's Guide to Quick Reporting

If scanning the QR Code is not for you, there are other options. You can run a Quick Report Utilizing either your SD Card or your iCode.

- » [Patient Instructions to run a Quick Report for Luna G3 Devices \(All models\)](#)
- » [Patient Instructional Video for Running a Quick Report utilizing your SD Card for Luna G3 Devices](#)
- » [Patient Instructional Video for Running a Quick Report utilizing your iCode for Luna G3 Devices](#)

## Understanding your Compliance Reports

After you have used one of the above methods to retrieve your report, or received your report from your DME or Physician, you may want to learn more about what it means. You can click the link below to learn more.

- » [Understanding your Compliance Report](#)



HERE'S TO A GREAT NIGHT'S SLEEP!



*\*\*Please note that React Health does not sell prescription devices directly to patients. We sell exclusively to DME Suppliers. Please do not contact React Health to obtain your Compliance Reports. Due to Patient Privacy Laws React Health cannot provide you with these reports. \*\**