



Modem PINs Quick Guide Luna II & Luna G3 PAPs

In order to facilitate remote device setting changes each PAP device equipped with a modem has a PIN number. This PIN number is unique to the device and modem.

Instructions for Luna II :

1. Plug the modem into the USB port on the back of the device
2. Power up the Luna II device
3. Once booted up, hold down the “RAMP” button until the Patient Comfort menu appears
4. Press the KNOB once to enter the menu
5. Turn the KNOB clockwise and scroll down to “ABOUT”
6. Press the KNOB once to enter the “ABOUT” menu
7. The last line of information displayed will be the unique PIN
8. Enter the PIN into the iCodeConnect patient database to enable REMOTE DEVICE SETTING changes

Instructions for Luna G3:

1. Power up the G3 device
2. Turn the KNOB until SETTINGS is highlighted
3. Push the KNOB once to enter the Patient Comfort menu
4. Turn the KNOB clockwise until you reach the last option on the second screen entitled “ABOUT”
5. Push the KNOB once to enter the ““ABOUT” ” menu
6. The fifth line of information (“ABOUT” halfway down the page) is the PIN
7. Enter the PIN into the iCodeConnect patient database to enable “REMOTE DEVICE SETTING” changes

**NOTE: changing either modem or PAP device will result in a new unique PIN. Further information “ABOUT” the iCodeConnect compliance database for 3B Medical, Inc. PAP devices can be found at www.iCodeConnect.com or by emailing info@3BProducts.com*