





For Emergency Use
Authorization Only.

Quick Operation Manual Luna G3 BPAP S/T & Luna G3 BPAP 25A

1. Button Functions


-  Home Button: Return to the previous menu or main interface
-  Knob: Adjust device settings
-  Start / Stop Button: Start / Stop delivering air

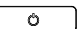
2. Setting up the Device

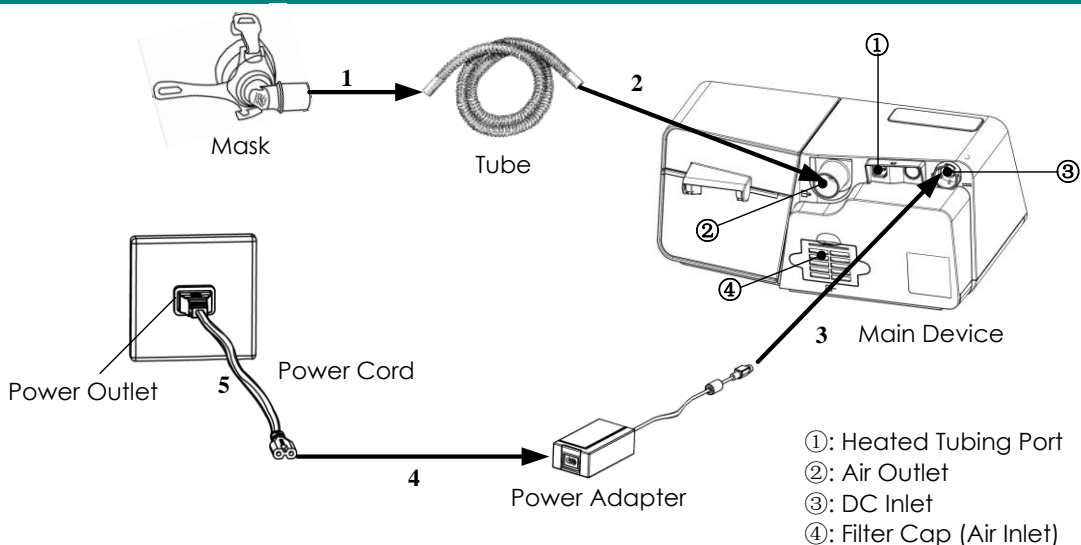


- A. Connect one end of the tube to the assembled mask.
- B. Connect one end of the tube to the air outlet of the device, as shown in the figure above. Or connect the heated tubing joint to the air outlet of the device, and then insert the power plug into the heated tubing port on the back of the device.
- C. Connect the power adapter to the DC inlet of the main device.
- D. Connect the power cord to the power adapter.
- E. Plug the power cord into the power outlet.

3. Adjusting Device Settings

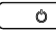
A. Connect the power cord and power adapter properly, The screen displays the Main Interface shown in the figure left below, press **the Start / Stop Button** , and the device will start delivering air, the screen displays the Main Interface shown the figure middle below (applicable to Luna G3 BPAP 25A model) or as shown in the figure right below (applicable to Luna G3 BPAP S/T model).

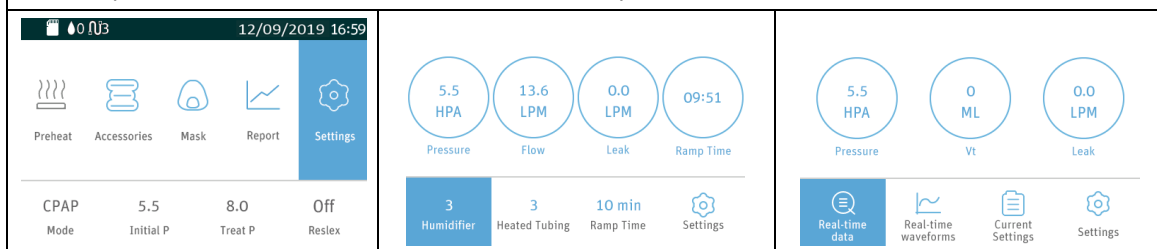


B. Turn **the Knob**  to access the Initial Setup Interface of the Patient Menu, as shown in the figure left below. The **Heated Tubing** option can only be adjusted when the device is connected to the heated tube, as shown in the figure right below.




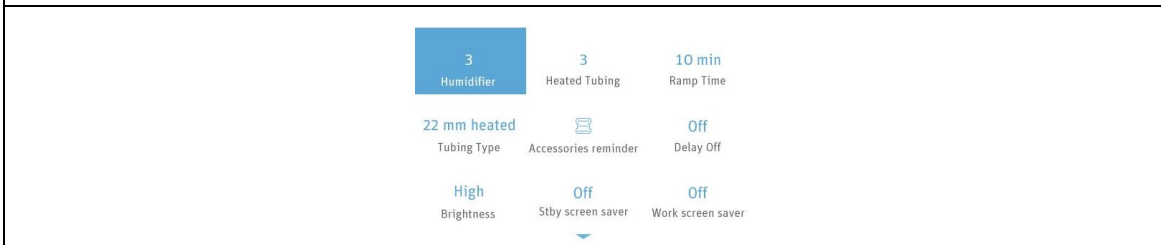
C. Press **the Knob**  to access the Setup Interface, as shown in the figure below.

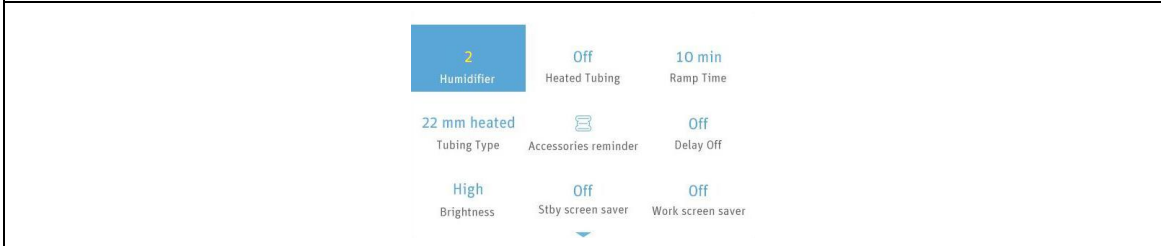

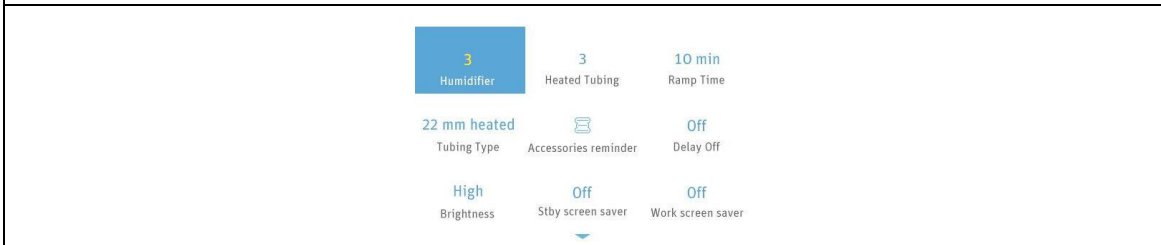


D. Turn **the Knob**  to adjust an option.




E. Press **the Knob**  to confirm your adjustment to the option.



F. The cursor switches among the options as you turn **the Knob** , as shown in the figure below. Refer to steps C-E to adjust other options. Press **the Home Button**  to exit the Setup Interface.

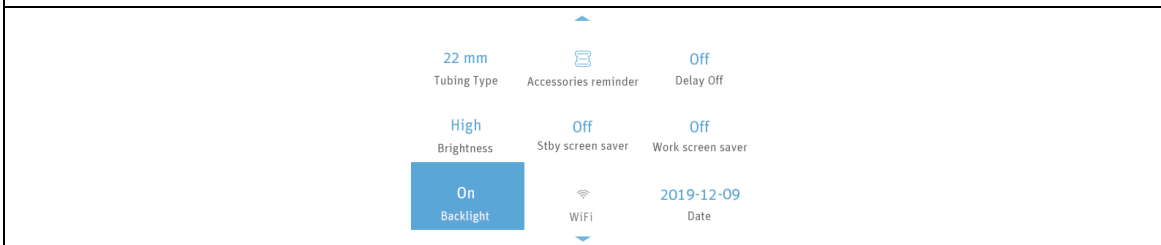

4. Starting Treatment

After you finish adjusting device settings and return to the Main Interface, press **the Start / Stop Button**  to start treatment.