


# CHANGING SETTINGS VIA SD CARD:

SD Card **MUST** have been previously utilized in the specific machine

The software for changing settings via SD card can be downloaded here:  
[https://3bproducts.com/pap\\_link.exe](https://3bproducts.com/pap_link.exe)

Once you have the software installed, the process for changing settings is:

- 
1. Insert the SD card into your computer's SD card reader.
  2. Open the PAP Link program.
  3. Click "Cancel" on the Patient List dialog that comes up immediately.
  4. Click the "Device Settings" button in the top bar.
  5. Navigate to the SD card if necessary, and select the file you want to change the settings for. Make sure the filename matches the serial number of the device (a serial number like ES412345678 will result in a filename of 12345678.USR).
  6. Change the settings as desired in the "Adjustment" column.
  7. Click "Confirm".
  8. Exit the program, then safely remove the SD card.

The next time this card is inserted into the machine, it will update its settings as it starts up.